



CROYDE MEDICAL

CROYDE TWIN COT

The Twin Cot is perfect for birth centres and postnatal & neonatal areas for twin infants.



The Croyde Twin Cot has been designed to create the best cot for newborn twins.

Its ergonomic design makes it easier for staff to care for twin infants as well as keeping mother and babies close.

Alongside its height adjustable frame, tilt function, removable mattress, and lowering side panels; the Twin Cot available with an optional basket for extra storage.



LOWERING SIDE PANELS

The easy-to-use release buttons on each side allow the side panels to be lowered for convenient access.

Designed with safety in mind, both catches must be released before lowering each side panel.

When lowered, the side panels provide full access along the side of the cot and enable it to be positioned next to the bed, allowing mothers to safely reach their babies.



HEIGHT ADJUSTABLE FRAME

The Croyde Twin Cot features a spring-assisted height adjustment pedal, allowing for raising and lowering in one smooth motion.

A clinician can adjust the height to their needs during neonatal examinations and observations, making work more comfortable.


Women who have had caesarean sections or complex can position their babies at a height that allows for comfortable and easy access.



TRENDELENBERG TILT

The Trendelenburg tilt function is easily activated by pulling the release knob at the end of the cot, allowing for a 15-degree tilt.

This tilt function helps support a newborn's delicate digestive system by keeping milk down and aiding the release of wind.

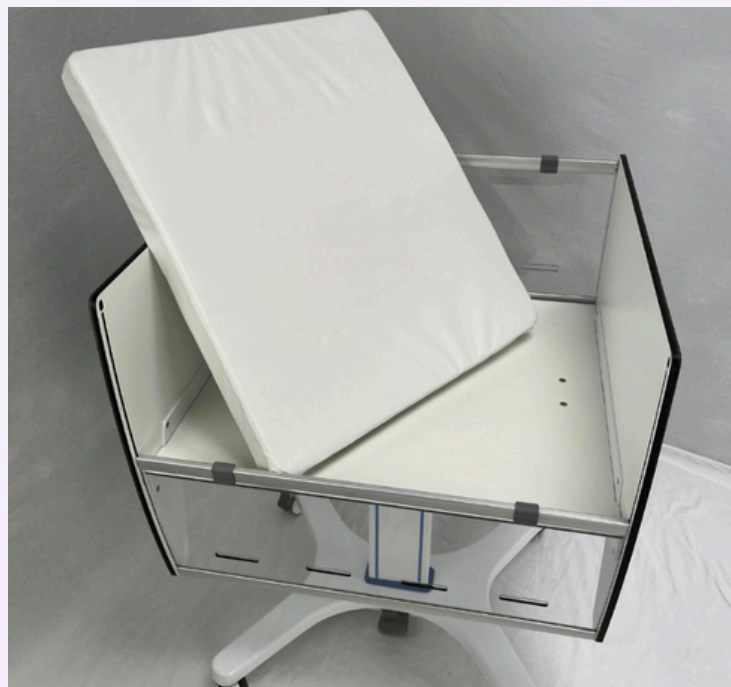

REMOVEABLE MATTRESS

The lightweight mattress is easily removed, providing full access to the cot deck for thorough cleaning.

The cot is compatible with hospital-approved wipes and chlorine-based cleaning solutions, making it suitable for use in high-traffic wards.

EVERYTHING YOU NEED TO KNOW

Click the links below to access the

Croyde Knowledge Hub

[Dimensions & Specifications](#)

[Delivery & Implementation](#)

[Cleaning Instructions](#)

[Warranty & Servicing](#)

[CLICK HERE FOR INFORMATION ON HOW TO BUY](#)



CROYDE MEDICAL

CONTACT US

Tel: 01367 241194

Email: info@croydemedical.co.uk

[**www.croydemedical.co.uk**](http://www.croydemedical.co.uk)

Croyde Medical, Unit A4 GBP1, Grove Business Park,
Downsview Road, Wantage, OX12 9FA

